

# PACS

سامانه Pacs در بیمارستان آیت اله کاشانی و هاجر شهر کرد

ارائه دهنده: اعظم فیاضی بروجنی

# سامانه Pacs در بیمارستان آیت اله کاشانی و هاجر

- استفاده از مرورگر Google chrome

- نحوه ورود به برنامه از داخل بیمارستان :

- روشن کردن Wifi گوشی

- آدرس 172.60.0.4

- آدرس 172.60.0.6

- نحوه ورود به برنامه از خارج بیمارستان :

- اتصال اینترنت گوشی

- 10.177.253.141

- 10.177.253.142

WEB DICOM VEIWER

Institution Name



Username



Password

SIGN IN

Image Server

About

Administration

METRIC V10.1

A True Web PACS Solution by [MARCO PACS](#)

- نام کاربری : **normal**
- رمز عبور : **normal**

# نمای برنامه پس از ورود به برنامه

BrowserExam

Not secure | 172.60.0.6/metric/Account/BrowseExam.html

marco pacs Public Favorite Tutorial About NORMAL

From 1399-6-26 To 1399-6-26 Today 1d 3d 1w 1m Any All CR CT DX DR MR SEARCH CLEAR

Review

54 studies /

Thumbnails	Patient Name	Patient Id	Farsi Name	Sex	Study Date	Age	Modality	Series	Images	Reported
	ahmadi^ali	k-890991			1399/06/26 09:56	26Y	CT	11	870	
	esmaillzadeh^ali	4621956035			1399/06/26 09:44	63Y	CT	4	135	
	TAHERI MOHAMMADMORAD	K-891166			1399/06/26 09:40	76Y	CT	2	52	
	hibatolah^kiani	6339489532			1399/06/26 09:40	92Y	CT	4	120	
	Test^1	99/06/26			1399/06/26 09:34	89Y	XA	1	1	
	ahmadi^kiamars	4610235137			1399/06/26 09:23	29Y	CT	4	114	
	solimani^azizgholi	5558877146			1399/06/26 09:17	70Y	CT	4	112	
	ASGARI+BAZALI	K-890439			1399/06/26 09:12	46Y	CT	2	26	
	asadi^shapor	4689306508			1399/06/26 09:09	41Y	CT	4	109	
	somaye-farajzade	4621751379			1399/06/26 09:09	38Y	DX	16	16	
	ASGARI^DAVUD	4610189828			1399/06/26 09:04	30Y	MR	8	54	
	AMIRI^BENYAMIN	466065189477202403			1399/06/26 08:47	16Y	MR	8	52	
	kiani^ehsan	4623338576			1399/06/26 08:43	34Y	CT	19	19	

Institution Name

© 2005-2019 MARCO PACS All rights reserved

V10.1

Type here to search

10:25 AM 9/16/2020

# امکان جستجو بر اساس تمام آیتم های زیر :

The screenshot displays the MarcoPACS BrowseExam web interface. The browser's address bar shows the URL `172.60.0.6/metric/Account/BrowseExam.html`. The interface includes a search bar with filters for 'From' (1399-6-26) and 'To' (1399-6-26), and a 'Today' button. Modality filters include 'All', 'CR', 'CT', 'DX', 'DR', 'MR', and 'XA'. A 'SEARCH' button and a 'CLEAR' button are also present. The interface shows '1 studies' and a 'Review' button. The table headers include 'Thumbnails', 'Patient Name', 'Patient Id', 'Farsi Name', 'Sex', 'Study Date', 'Age', 'Modality', 'Series', 'Images', and 'Reported'. There are three search input fields below the headers.

# امکان جستجو بر اساس نام بیمار و یا شماره سریال پذیرش بیمار

The screenshot displays the MarcoPACS web interface. At the top, there is a navigation bar with 'Public' and 'Favorite' tabs, and a search bar. Below the navigation bar, there are filters for 'From' and 'To' dates, and a 'Today' button. The main content area shows a table of search results with columns for Patient Name, Patient Id, Farsi Name, Sex, Study Date, Age, Modality, Series, Images, and Reported. The table contains six rows of data for patients with names starting with 'ahmadi'. At the bottom of the interface, there is a footer with 'Institution Name', copyright information, and a Windows taskbar.

Thumbnails	Patient Name	Patient Id	Farsi Name	Sex	Study Date	Age	Modality	Series	Images	Reported
	ahma									
+	ahmadi^ali	k-890991		♂	1399/06/26 09:56	26Y	CT	11	870	
+	ahmadi^kiamars	4610235137		♂	1399/06/26 09:23	29Y	CT	4	114	
+	ahmadi^zahra	1757512179		♀	1399/06/26 09:35	33Y	CT	4	110	
+	ahmadi^nasim	4610362619		♀	1399/06/26 09:31	28Y	CT	4	110	
+	ahmadi^zohreh	4610174529		♀	1399/06/26 09:26	30Y	CT	4	107	
+	ahmadi^javad	k-891214			1399/06/26 08:31		DX	3	3	

# امکان جستجو بر اساس نوع کار ( آنژیوگرافی / سی تی اسکن / ام آر آی / رادیولوژی )

The screenshot displays the MARCO PACS BrowseExam web application. The interface includes a search bar with filters for 'From' (1399-6-26) and 'To' (1399-6-26), and a 'Today' button. Modality filters include All, CR, CT, DX, DR, and MR. A search button and a 'CLEAR' button are also present. Below the search bar, there is a 'Review' button and a '1 studies /' indicator. The main area contains a table with columns for Thumbnails, Patient Name, Patient Id, Farsi Name, Sex, Study Date, Age, Modality, Series, Images, and Reported. The table shows one study for patient 'ahmadi^javad' with Patient Id 'k-891214', Study Date '1399/06/26 08:31', Modality 'DX', Series '3', and Images '3'. The bottom of the screen shows the Windows taskbar with the search bar, taskbar icons, and system tray.

Thumbnails	Patient Name	Patient Id	Farsi Name	Sex	Study Date	Age	Modality	Series	Images	Reported
+	ahmadi^javad	k-891214			1399/06/26 08:31		DX	3	3	



# کلیک روی نام بیمار جهت مشاهده کلیشه بیمار

BrowseExam x rezayi\*dariush x +

Not secure | 172.60.0.6/metric/Account/ShowStudy.html?UID=1.3.12.2.1107.5.1.4.69623.30000020091607431206200000092&SPName=

**MARCO PACS**  
Web DICOM Viewer

rezayi\*dariush / CT / ...

HAJAR HOSPITAL(SHAHREKORD)  
SE: 1  
IM: 2 of 1

rezayi\*dariush  
Age: 040Y  
M  
20.09.16-12:06:10-DST-1.3.12.2.1107.5.1.4.69623  
Study Date: 1399/6/26

1 Topogram 0.6 T20s  
HRCT\_IN\_corona  
CHEST

2 ThorHR IN 5.0 B90s  
HRCT\_IN\_corona  
CHEST

3 ThorHR IN 1.5 B80s  
HRCT\_IN\_corona  
CHEST

501 Patient Protocol

Thickness: 0.6  
Zoom: 1.36  
WW:350 WL:50

Activate Windows  
Go to Settings to activate Windows.

© 2005-2019 MARCO PACS All rights reserved

Type here to search

12:30 PM  
9/16/2020